

## STATEMENT OF LIMITED WARRANTY

### 1. WARRANTY EXCLUSIONS

The foregoing warranty does not cover and shall be void for defects or damages resulting from casualty, accident, misuse or abuse, neglect, alterations, service or repair by other than by Doty Belt or its Independent third party service providers, Improper use (including without limitation any use as a rappelling harness) or other causes not arising out of defects in the materials or workmanship of Products.

### 2. PROCEDURE FOR WARRANTY SERVICE

Prior to returning any Product to Doty Belt, you must first contact Doty Belt, either by telephone (to (916) 698-2551) or email (support@dotybelt.com), to confirm that your Product is covered by the foregoing warranty. To do that, you may be required to produce a copy of the original receipt or other proof of purchase from Doty Belt. If such Product is covered, you must obtain from Doty Belt a return merchandise authorization ("RMA") number and shipping instructions prior to any shipment of such Product to Doty Belt, and include that MA number with any corresponding Product shipment. Where a Product is not covered by the foregoing warranty, Doty Belt will agree to sell a replacement Product to you at a price discounted from the then-current list price for such Product, depending on how long you have owned the Product, as follows:

Product has been owned for less than six (6) months	Twenty- five percent (25%) discount
Product has been owned six (6) months or more, but less than one (1) year	Twenty percent (20%) discount
Product has been owned for one (1) year or more, but less than eighteen (18) month	Fifteen percent (15%) discount
Product has been owned for eighteen (18) months or more	Ten percent (10%) discount

### 3. WARRANTY DISCLAIMER

THIS STATEMENT OF LIMITED WARRANTY IS THE SOLE WARRANTY MADE BY DOTY BELT WITH RESPECT TO PRODUCTS OR ANY MATTER WHATSOEVER.

TO THE EXTENT PERMITTED BY LAW, DOTY BELT HEREBY DISCLAIMS ALL OTHER WARRANTIES, INCLUDING WITHOUT LIMITATION ANY IMPLIED WARRANTIES OF FITNESS FOR A PARTICULAR PURPOSE, MERCHANTABILITY OR NONINFRINGEMENT OF THIRD PARTY RIGHTS. Some jurisdictions do not permit the disclaimer of certain warranties so the foregoing may not apply to you.

### 4. RETURN

If you do not agree to the terms of this Statement of Limited Warranty, you may return the Product to Doty Belt, unused and in its original packaging, within ten (10) days of purchase for a full refund.

If you have any questions about the Product or this Statement of Limited Warranty, you may contact Doty Belt at:

Doty Belt, LLC  
8260 Brianna Court  
Sacramento, CA 95829  
Attention: Customer Service

Phone: (916) 698-2551  
Fax: (916) 647-9700  
[gdoty@dotybelt.com](mailto:gdoty@dotybelt.com)

DOTY BELT and the Doty Belt logo are trademarks of Doty Belt, LLC and may not be used without prior written permission.



## PATIENT TRANSFER SLING



WARRANTY & RETURN POLICIES  
USER INFORMATION



**DOTY BELT, LLC**  
8260 Brianna Ct.  
Sacramento, CA 95829  
916-698-2551  
[WWW.DOTYBELT.COM](http://WWW.DOTYBELT.COM)

# IMPORTANT USER INFORMATION PLEASE READ AND SAVE

The Doty Belt™ Patient Transfer Sling is a universal patient transfer sling with multiple uses and applications. It supports the weight of the patient while eliminating potential injury for the caregiver. The vinyl backing is not only strong and wide enough for patient comfort, but can easily be wiped clean with a sanitizing agent. Hand grips on either side of the Sling provides a variety of working positions and adjustments. The Sling is easy to carry and store.

## DONNING INSTRUCTIONS

Simply wrap the yellow portion of the sling around the back of the person and maneuver handles under the arms. Adjust handles as needed.

### STEP 1



### STEP 2



### STEP 3



### STEP 4



**SAVE YOUR BACK - SAVE YOUR CAREER**



## KEY FEATURES

- Formed rubber grips for user comfort
- Lift Handles Extend to 36 Inches
- PVC laminate over Polyester for ease of cleaning and sanitation
- Adjustable cam lock for ease of handle adjustments
- Polyester webbing construction for durability and strength
- 30 Inch by 12 Inch backing for patient comfort
- Third handle to assist in lifting
- 22kn rated aluminum carabiner
- 300 lb weight limit

## THE PTS CAN BE USED FOR:

1. Lifting a person off the floor to a Bed, Gurney, or Chair.
2. Sliding over to a hospital bed from a gurney.

## INSPECTION

Inspect Patient Transfer Sling for frayed, cut or worn material. Look for excessive chaffing at the wear points; typically where the polyester webbing comes into contact with the carabiner.

## REPAIR

Doty Belt requires that all repair work be done by the manufacturer. All other repair work or modifications may void the warranty and shall release Doty Belt, LLC from all liability and responsibility as the manufacturer.

## MAINTENANCE / CLEANING

The Patient Transfer Sling can be wiped down with a sanitizing agent. Use the cleaning agent per manufactures recommendations. The sling can also be clean with a mild detergent. Clean and dry this equipment after each use to remove any dust, debris and moisture. Do not put in a dryer. Hang dry out of direct sunlight. Do not use bleach.