

# Guidelines for Use

## Taylor Spit Hood – Item # TAY-SH285W

Disposable - ONE TIME USE

### Read Before Using

**Warning – Improper use can lead to suffocation or asphyxiation.**

#### Purpose:

- To stop or reduce a person's oral/nasal/facial fluids from becoming airborne and coming into contact with another individual.

#### Guidelines:

- Carefully place the disposable spit hood over the head of the individual, while making sure it fits well without causing unintended discomfort.
- Check that the disposable spit hood is secure, allowing for full breathability while preventing spitting.
- Do not leave the individual unattended and continually monitor to ensure their safety while the disposable spit hood is in use.
- When safe to do so, carefully remove and properly dispose of the spit hood.

